

# HeartSafe® Digital Combination Lock Cabinet

G3 Design Cabinet from July 2015 (Serial number: 47296-024 on)



AED Locator and Communities working together to make the UK

# HeartSafe®

## HeartSafe® Digital Combination Lock cabinet

Highest quality AED cabinet designed with enhanced security features to hold your defibrillator ready for public access in an emergency. Fully featured to store your AED in a temperature controlled environment with extremely low power consumption and simple installation.

Access in an emergency is simple, intuitive and above all fast, saving every vital second of time in getting a defibrillator to the patient.

All HeartSafe® AED Locator cabinets benefit from our full governance and management oversight systems. We are supported by all of the regional Ambulance Service Trusts.

A combination of what we believe to be the best cabinet on the market and defibrillators designed specifically for laypeople with or without training, the HeartSafe® system is proven to help save lives.

- Highly visible community presence of accessible AED
- Highly Secure, high quality **Stainless Steel** construction
- 10 Year anti-corrosion warranty on cabinet body
- Fully **CE Marked** and approved
- Access controlled via 4 digit PIN code
- Simple to use quality electronic lock with visual and audible feedback
- Integrated green dusk-to-dawn LED Beacon
- External courtesy light over lock to aid accessibility at night
- Internal courtesy light
- Thermostatically controlled heater
- Simple installation, very low power and running costs



## High quality electronic combination lock


- Quick and very easy access compared to mechanical locks
- Very low effort push buttons
- Audible 'beep' confirmation for each button press
- LED to indicate good/bad access code entry
- Key override feature




Dusk to dawn LED courtesy light over pushbutton lock with PIR on/off proximity sensor



## Ambulance Service Control Centre

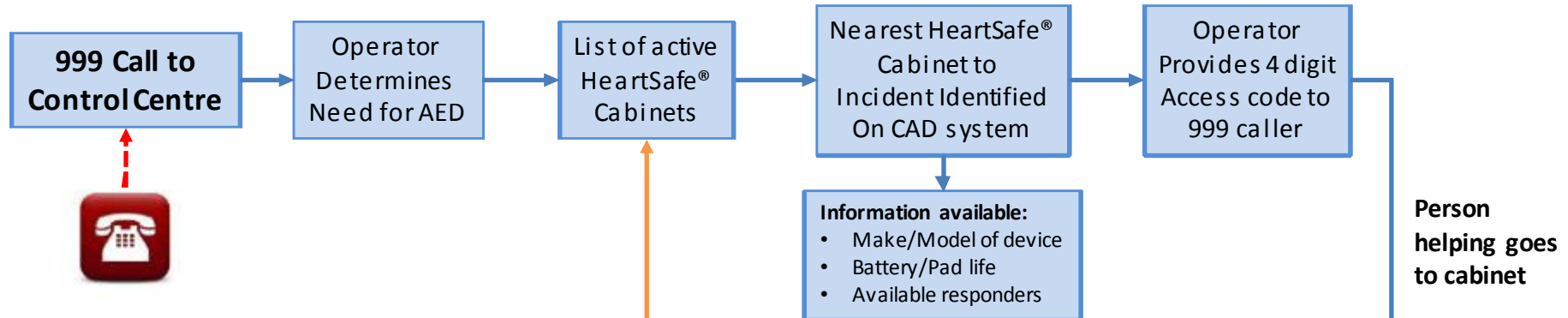


## AED Locator Network

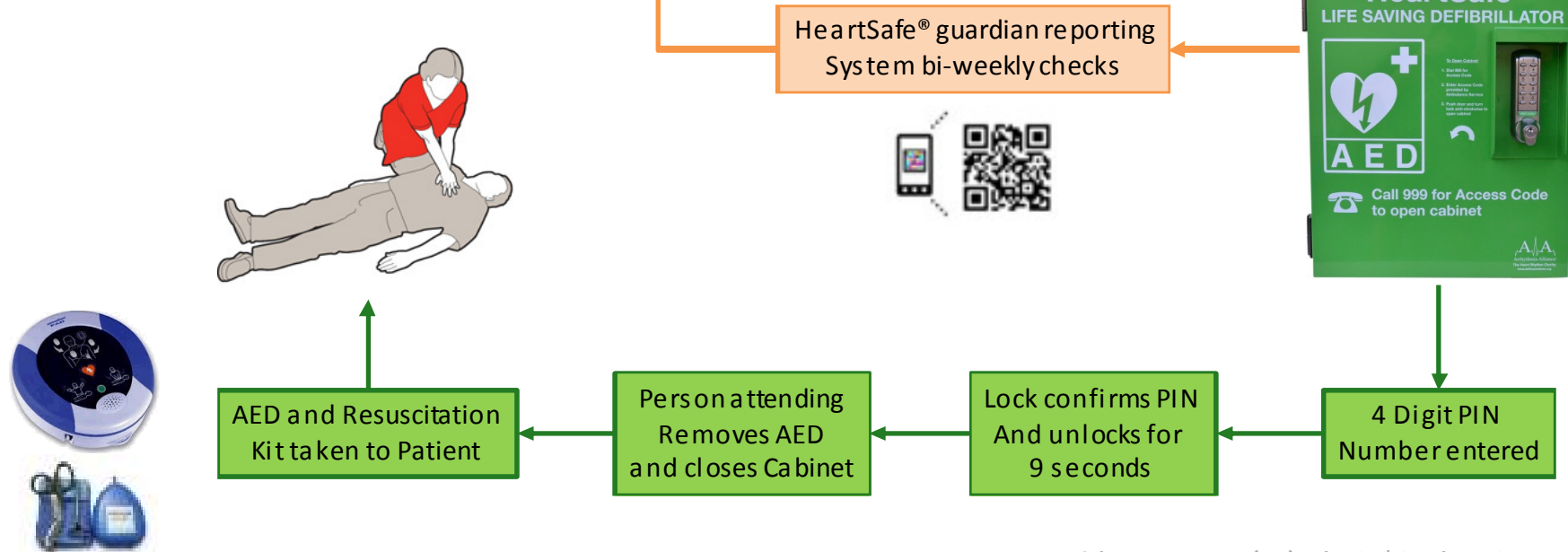

## Cabinet internal view (Shown with HeartSine® Samaritan PAD 500P)

- All stainless steel construction, heated and insulated cabinet
- Large enough to accommodate all common AED's

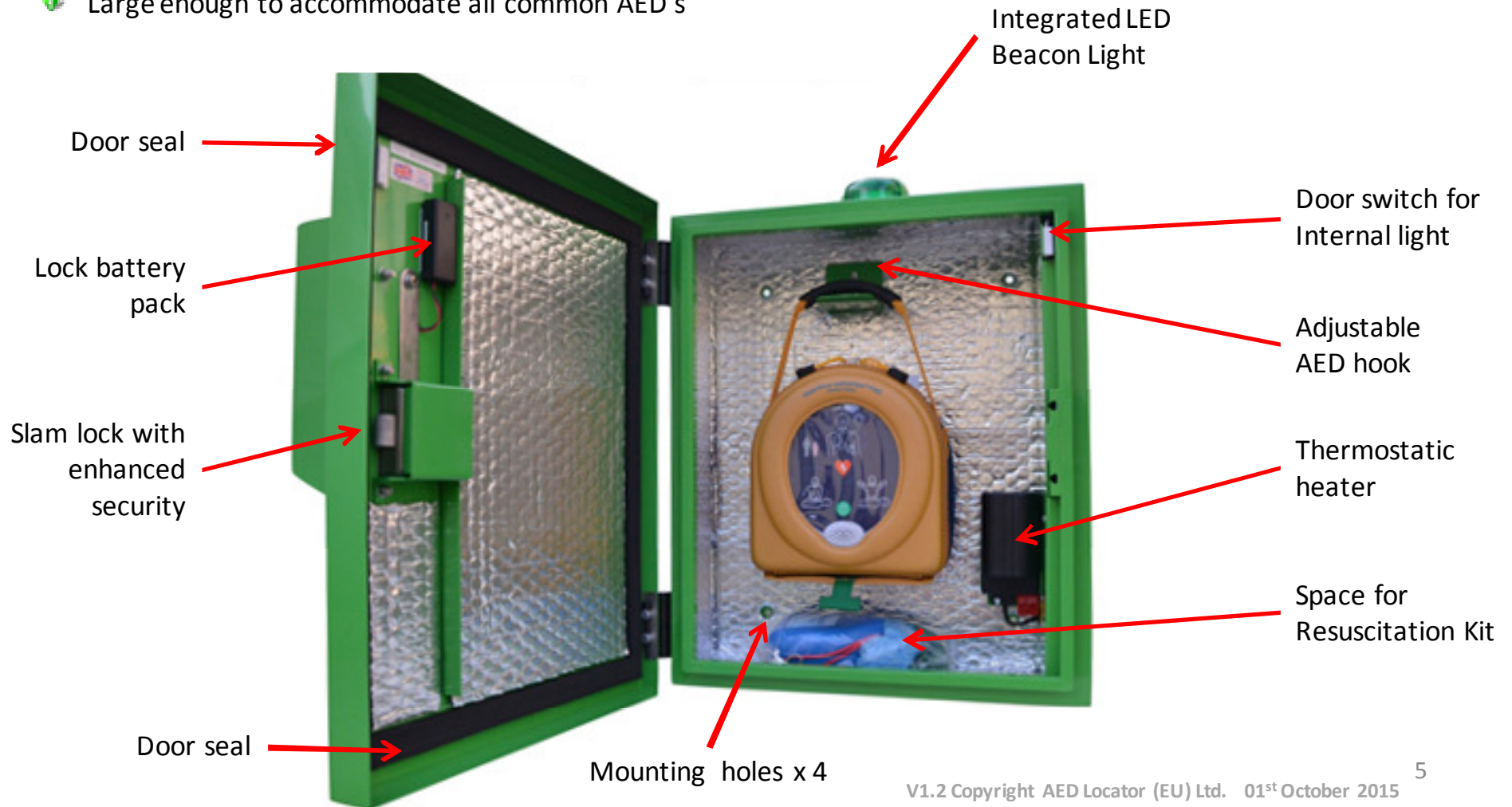

HeartSafe® Cabinets house all common Defibrillators, including:



## Cabinet Technical Data

- 📏 Dimensions Overall (HWD): 530 x 420 x 240mm (Incl. lock shroud and Beacon)
- 📏 Dimensions Cabinet (HWD): 500 x 420 x 175mm
- 📏 Dimensions (Incl. Packaging): 540 x 500 x 300mm (Shipping carton)
- 📏 Cabinet weight (empty): 12.8Kg (28lbs)
- ⚡ Electrical Requirements: 120-240VAC, 50-60Hz, 100W (Max)
- 🏠 Cabinet material: Grade 304 Stainless Steel
- 🎨 Green paint finish: Polyester Powder Coat
- 🔒 Cabinet IP rating: IP54 (minimum)
- 🔥 Heater: 100W touch safe thermostatically controlled
- 💡 Cabinet internal light: 3W LED strip (only operates when door is open)
- 💡 External lock illumination: LED with PIR switch (active dusk to dawn only)
- 💡 External LED Beacon light: Integrated Green LED with dusk to dawn control
- 🔑 Push button Lock: Proprietary digital electronic lock with audible and visual feedback. Manual key override
- 📄 Approvals: CE, 2004/108/EC, 2006/95/EC, 2011/65/EU
- 📄 **Supplied with full installation template, instructions, warranty and certification**



# HeartSafe® Cabinet Warranty Information

## Component Warranty Detail

📌 Cabinet body:	10 Year anti-corrosion, excluding paint finish and screen printing
📌 Internal Electrical Components:	3 Years (includes: heater, PCB, internal light, power supply, door switch)
📌 Digital Electronic Lock:	2 Years
📌 Lock courtesy light:	2 Years
📌 External LED Beacon light:	2 Years

**AED Locator warrants that the HeartSafe® cabinets sold in accordance with published specification and free from defects in materials or workmanship according to the periods as detailed above. Warranty dates from the date of delivery of the cabinet to the customer.**

Warranty is applicable if used under normal circumstances and provided any defects or failures are not the result of:

1. Installation in a manner other than as per our manual or other specific instructions
2. Customer or third-party abuse or misuse
3. Failure to adhere to HeartSafe® AED Locators instructions regarding ongoing maintenance and care
4. Except as otherwise stated in AED Locator product literature, wear or deterioration due to environmental conditions or abuse

This is the sole and exclusive warranty of HeartSafe® AED Locator; there are no other warranties, either express or implied, as to any matter whatsoever, including, without limitation, the implied warranties of merchantability or fitness for particular purpose. The sole and exclusive remedy of the customer for any breach of warranty by AED Locator, shall be that we provide upon confirmation of the defective condition of the subject part at no cost to the customer a replacement part. Fitting or installation of any replacement parts shall be the responsibility of the customer.

In no event shall HeartSafe® AED Locator be liable for consequential or indirect damages or economic losses, loss of use, lost profits, downtime or damages due to delay, whether by reason of breach of warranty, breach of contract, negligence, strict liability or otherwise, whether foreseeable or not and whether or not HeartSafe® AED Locator has been advised of the possibility of such damage.



**For any more questions regarding HeartSafe® Cabinets or general questions on AED Locator or advice on Public Access Defibrillation  
Please contact...**

AED Locator (EU) Ltd.

Swallet Farm

Old Bristol Road

East Harptree

Bristol

BS40 6DQ

Email: [enquiries@aedlocator.org](mailto:enquiries@aedlocator.org)

Tel: 01275 332323

 @AEDLocator